

# Abilitations®

## CuddleLoop

Guide to Accompany Item #030940

### Introduction

Welcome to the Abilitations CuddleLoop, the best thing around a body besides a hug! Inspired by a therapist, the CuddleLoop gives a tight, snugly swaddle effect for just the right amount of sensory input. Designed for children with sensory processing disorders, the CuddleLoop provides deep touch pressure to help with calming and organizing. Perfect for circle time, story time or anytime extra touch pressure, calming and defining spatial boundaries is needed.

### Product Use

Place the child in the center of the CuddleLoop fabric and pull snugly around the back and/or hips, thighs and leg area. Secure with a knot or use a fastener (such as a hair clip or chip clip, not included).

### Care & Safety

The CuddleLoop should not be used as a restraining device. Use with adult supervision at all times and allow for frequent breaks from the CuddleLoop to ensure maximum safety and effectiveness. Recommended wearing schedule may be determined by the therapist according to each child's sensory needs. The CuddleLoop is a 24"W x 60"L (61cm x 152cm) circular piece of lycra fabric designed to wrap around the child's body. Colors may vary. Machine wash on gentle cycle and hang dry or tumble dry on low.



### Recommended Products

If you like our CuddleLoop, check out these other great Abilitations products:  
**Abilitations Chair Hugger:** 031048



©2013 School Specialty • PO Box 1579, Appleton, WI 54912-1579 • USA • All rights reserved.

Customer Service 1-888-388-3224 • [www.schoolspecialty.com](http://www.schoolspecialty.com)

Or reach us via e-mail at: [orders@schoolspecialty.com](mailto:orders@schoolspecialty.com)

**1-YEAR GUARANTEE OF SATISFACTION**

See more fun activity guides for other great  products at [www.abilitations.com](http://www.abilitations.com).